**INSTRUCTIONS FOR USING
INSTRUMENTS / CASSETTES**

EN.	2
UA.	2
BUL.	3
HUN.	3
ELL.	4
DAN.	5
GLE.	5
SPA.	6
ITA.	6
LAV.	7
LIT.	7
MLT.	8
DEU.	8
NLD.	9
POL.	9
POR.	10
RON.	10
SLO.	11
SLV.	11
FIN.	12
FRA.	12
HRV.	13
CZE.	13
SWE.	14
EST.	14
GEO.	15
KAZ.	16

INSTRUCTIONS FOR USING INSTRUMENTS / CASSETTES

Please read the instructions carefully before using the instrument.

1. FIRST USE (INITIAL STEP)

The instruments are delivered "non-sterile": thoroughly wash, rinse and sterilize them before use. Most of the connected instruments contain oily residues in the parts as a result of the manufacturing steps; please make sure the instruments are properly cleaned.

If the use of a washing machine is not possible, the instruments can also be washed by hand: use a stiff brush with plastic bristles (NOT metal bristles), and always use appropriate cleaning and degassing solutions.

2. STERILIZATION

We recommend sterilizing the instruments only after proper cleaning. If the instrument is not properly cleaned, sterilization will not be effective. Please follow the instructions for use provided by the manufacturer of the sterilization equipment.

There are different ways to sterilize the instruments:

2.1 AUTOCLAVE STERILIZATION

The main purpose of autoclave sterilization is to achieve proper sterilization while preserving the structural and functional characteristics of the instruments as much as possible.

Only experienced personnel who underwent specific training must perform this process.

- All cleaning operations must be carried out only in places properly equipped and arranged for such operations.
- Immediately after use, the instruments must be disinfected using solutions with proved properties. Be sure to use the correct dosage of cleaning / degassing solutions, duration of cleaning and temperature mode; always follow the instructions by the equipment manufacturer and cleaning solution manufacturer.
- Cleaning solutions / disinfectants that contain sodium chloride (NaCl) can often cause oxidative corrosion of the instruments, affecting their further use.
- Cleaning / disinfectant solutions must be changed daily.
- Do not put the instruments made of different metals in the same trays for washing / sterilization.
- Connected instruments must be in the open position for cleaning / disinfection.
- Make sure the trays are not overloaded with instruments, since this can reduce cleaning / disinfection efficiency.
- Possible residues after the first cleaning must be removed.
- The temperature for the final rinse has been proven to be between 80°C and 90°C.
- It is strongly recommended to use only demineralized water for the final rinse. This helps to avoid the dark spots.

Autoclave sterilization using saturated steam is the most commonly used method.

- Do not exceed 177°C. Steam sterilize for at least 4 minutes at 132°C or for 30 minutes at 121°C. A 30-minute drying time is recommended after the sterilization cycle. Follow the manufacturer's instructions for proper operation of the sterilizer. Place the wrapped cassettes in the sterilizer, making sure the cassettes are arranged at a certain distance. Follow the instructions of the sterilizer manufacturer for the packaged instruments to determine cycle time, temperature, and drying instructions. Remove the cassettes. Make sure that the control tape on the outside of the package or the indicator / integrator inside the package has changed its colour (when a steam autoclave is used).
- For a packaged instrument, it is very important to complete the rinse cycle thoroughly, since if there is some water left in the envelope, it can cause corrosion or oxidation of the instruments and cause the bacteria growth.
- Sterile instruments must be stored in specific, dry and dust-free places.

WARNING: DRY HEAT STERILIZATION

Sterilization at above 177°C reduces the strength of the instrument, while also causing darkening of the instruments, developing of brown spots that look like oxidation.

THE MANUFACTURER WAIVES ANY LIABILITY IN THE FOLLOWING CASES:

- Damage caused by improper use of the instrument;
- Damage caused by an incorrect process of cleaning and/or sterilization of the instrument;
- Damage caused by storing the instruments in inappropriate places;
- Damage caused by lack of instrument maintenance, especially for joints that require regular lubrication at the joints.

RECOMMENDED PROCESSING METHODS**CLEANING****Ultrasound**

- Follow the manufacturer's instructions for proper operation of the ultrasonic cleaner / disinfectant.
- Meddins recommends 16 minutes as a minimum cleaning time. Actual cleaning time / disinfection may vary.
- Clean Meddins instruments or cassettes in a neutral pH enzymatic cleaner or multi-purpose cleaner / disinfectant.
- After ultrasonic cleaning, thoroughly rinse the cassette with demineralized water to remove detergent or particles.

Mechanical / Automatic cleaner

- Follow the manufacturer's instructions for proper cleaning in an automatic cleaner.
- Clean Meddins instruments and cassettes only using detergent/disinfectant with a neutral pH.

Thermal disinfectant

- Follow the manufacturer's instructions for proper cleaning and disinfection.
- Clean Meddins instruments and cassettes only using detergent / disinfectant with a neutral pH.

INSPECTION / PREPARATION

Open the cassette and inspect the instruments, making sure all dirt has been removed from the instruments and cassette during ultrasonic cleaning. Check for broken or missing instruments. The instruments must be dry before packaging or sterilization, to prevent moisture accumulation in the package. Place the cassette in self-sealing sterilization bags.

STERILIZATION

For all cassettes - Steam autoclave. Do not exceed 177°C. Steam sterilize for at least 4 minutes at a 132°C or for 30 minutes at 121°C. A 30-minute drying time is recommended after the sterilization cycle. Follow the manufacturer's instructions for the correct operation of the sterilizer. Place the cassettes in the sterilization bags into the sterilizer, making sure that the cassettes are arranged at a certain distance. Follow the instructions of the sterilizer manufacturer for the packaged instruments to determine cycle time, temperature, and drying instructions. Take out the cassettes.

WARNING

All Meddins stainless steel cassettes must not be exposed to the following chemicals:

- | | | |
|--|--------------------------|-------------------------|
| • Activated perchloric acid / chlorine dioxide | • Potassium permanganate | • Citric acid |
| • Barium chloride | • Sodium hypochlorite | • Iron chloride |
| • Calcium chloride | • Tartaric acid | • Mercury chloride |
| • Chlorinated lime | • Iron chloride | • Phenol |
| • Dakin's solution | • Hydrochloric acid | • Potassium thiocyanate |
| • Lysoyl | • Aluminium chloride | • Tin chloride |
| • Mercury salts | • Carbolic acid | • Sulphuric acid Iodine |

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